



GlucoDr. auto™ A - Blood Glucose Monitoring System


Compact, simple, precise. Results in just 5 seconds.

Diabetes


Modern living has spurred an explosion in the development of diabetes mellitus type II in later life. As with diabetes mellitus type 1, management of the condition requires the self testing of blood glucose levels by the patient up to several times a day. For this reason, small, portable and simple testing kits can be a great help.

- ✓ Measurement in just two steps
 - Insert test strip and check the code number
 - Apply blood drop to the strip
- ✓ Requires only 0.5 µL of blood
- ✓ Accurate results in 5 seconds
- ✓ Automatic calibration




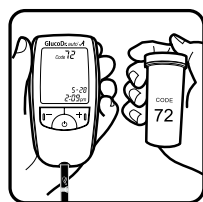

Insert test strip and check the code number

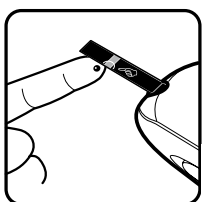

Apply a drop of blood to the side edge of the test strip. Wait for the drop to completely fill the yellow window of the test strip. The meter will then begin the count-down.

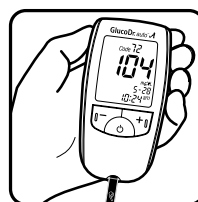

The blood glucose result will be displayed on the meter in just 5 seconds.

Technical data

Model	GlucoDr. auto™ A Blood Glucose Monitoring System AGM-4000
Assay Method	Electrochemical method
Sample Type	Fresh capillary, venous whole blood
Sample Volume	0.5 µL
Measurement Range	20-600 mg/dL (1.1-33.3 mmol/L)
Measurement Time	5 seconds
Calibration	Plasma-equivalent
Unit of Measurement	mg/dL or mmol/L
Operating Ranges	Temperature: 10-40 °C, Relative Humidity: Less than 85%, Altitude: Up to 2500 m, Hematocrit: 20-60%
Dimension	93.5 x 49.0 x 17.5 mm
Weight	Approximately 40 g (with battery)
Battery	1x CR2032 for about 1,000 tests
Display	37.0 x 42.0 mm LCD
Memory Capacity	500 test results (with date, time and event)
Data Transmission	USB

Product overview

Product	Product Code	Tests / Pack
GlucoDr. auto™ A AGM-4000 Test Meter	9x0x01	1 kit (test meter, test strips, carrying case, user manual, battery, lancing device, lancets)
GlucoDr. auto™ A Test Strip	9x0x02	2 x 25 strips
GlucoDr. auto™ Control Solution	9x0x03	2 x 4 mL (1x normal 130 mg/dL and 1x high 320 mg/dL)
GlucoDr.™ Lancing Device	9x0005	1 lancing device
GlucoDr.™ Lancets	9x0006	200 lancets

Our **customer service team** is on hand to answer any questions you may have by telephone, via email or in a personal consultation. For details on **your local sales office and regional free phone numbers**, visit www.nal-vonminden.com.

We help you care!

Rapid Tests

Laboratory Diagnostics

Laboratory Service

Consulting & Service

Headquarters

nal von minden GmbH · Carl-Zeiss-Strasse 12 · 47445 Moers · Germany
 Chief Executive Officers: Sandra von Minden · Roland Meißner · Thomas Zander
 Commercial reg. Kleve · HRB 5679
 Tel.: +49 941 290 10-0 · Fax: +49 941 290 10-50

info@nal-vonminden.com
www.nal-vonminden.com