

### new art laboratories

#### Now available:

- ✓ First and only mobile cup test reader
- ✓ Versatile: For use with Drug-Screen and medical tests in dip, cup and cassette format
- ✓ Premium high-tech smart device with long battery life
- ✓ Data capture using innovative and intuitive software
- ✓ Highly accurate evaluation in just a few seconds



# RSS flash mobile reader

Test, interpret, document The complete system for rapid tests



### **RSS flash**

Whatever your field of operation, the new RSS flash enables the evaluation of nal von minden rapid tests in the blink of an eye.

This compact device combines with easy-to-use software to make for the ultimate user-friendly experience. Thanks to our magnet adapter system, the device can evaluate single tests, multi-cassette tests and even cup tests. Whether you are looking to detect drugs in urine or a medical parameter in blood or serum - the RSS flash provides you with a precise and objective interpretation in no time at all!

and documentation of results has reached a new level, thanks to

instant data capture using our innovative and intuitive software.

- Easy to use
- Made from robust materials
- Simple and objective evaluations in seconds
- Automatic barcode recognition
- Software in multiple languages
- Portable for flexible implementation
- Performance not adversely affected by poor lighting conditions

- Evaluation of up to
  12 parameters simultaneously
- Complete documentation of results
- Individual customer support
- Data export via CSV



## Multi-talented – Objective and precise results in seconds.

A portable diagnostic device that delivers objective and precise results in a flash! Robust and lightweight, it can be used anywhere. It delivers clear and simple results in just 10 seconds, regardless of external conditions. A reliable interpretation of cassette- and cup tests results is guaranteed. The visualisation

Your options

- Drug-Screen<sup>®</sup> Single cassette tests
- ✓ Drug-Screen<sup>®</sup> Multi-dip- / Multi-cassette tests
- Drug-Screen® Cup II
- Drug-Screen<sup>®</sup> Saliva Cup
- Tracedetect tests
- NADAL<sup>®</sup> Medical rapid tests

#### Various sample materials:


Urine, saliva, whole blood / serum / plasma, solids

Barcoded tests for fully automated test recognition and accurate analysis



### Test & interpret -Reliable results in just 3 steps

#### Procedure

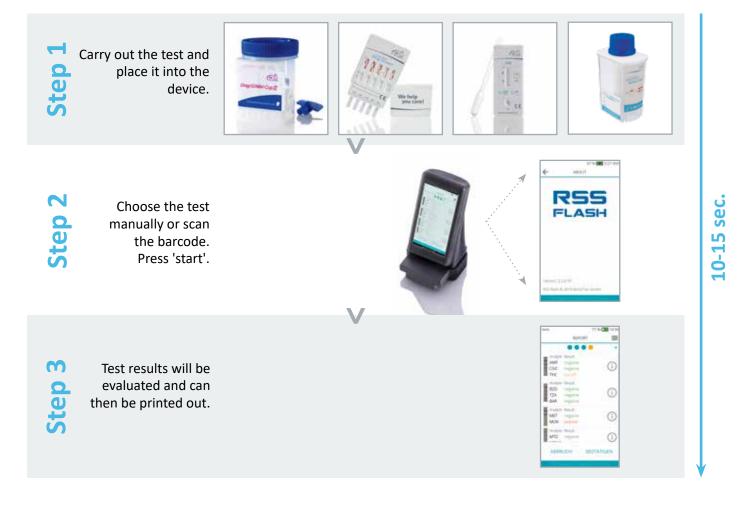

#### Result

The results are clearly displayed and can be saved or printed out easily. Furthermore, all results and images can be transferred to a PC once the device is connected. With ongoing updates, your device can always be state of the art!

nvm	77 % 16:50	
	REPOR	r 🔳
		• v
analyte	Result	
AMP	negative	$\bigcirc$
COC	negative	U
THC	positive	
Arialyte	Result	
BZD	negative	$\bigcirc$
TZA	negative	Û
BAR	negative	
Analyte	Result	
MET	negative	(i)
MOR	positive	0
Analyte	Result	-
MTD	negative	(i)
		0
ABBR	UCH	BESTÄTIGEN
		BESTÄTIGEN

#### Print results

Using the optional cable-free bluetooth printer (available with a paper width of 5.6 cm or 8 cm), you can print results on the spot.



### **Technical data**

Features		
Number of tests per analysis	1 cassette test or 1 cup test	
Device dimensions (770070)	850 x 145 x 550 mm	
Weight	315 g	
Display	4.6" (11.7 cm)	
Battery life	8-10 hours	
Power source	1x external power cable, CE-certified	
Input	100V - 240V, 50 - 60 Hz, 0.6 A	
Output	5V, 1500mA	
USB cable length	1 m	
Humidity	20% - 80%, non-condensing	
Operating temperature	-10 – +45 °C	
Operating altitude	0 - 2000 m above sea level	

### **Order information**

Product	Description	Product code
RSS flash	Mobile analyser, USB cable, power cable, multifunctional device, user hand- book, software download link	770070
RSS flash Cup	Mobile analyser, USB cable, power cable, cup adapter, user handbook, software download link	770071
RSS flash transport case	Practical transport case for reader and accessories	770073
Upgrade kit: cassette	Multi-cassette adapter, single	770078
Upgrade kit: cup	Cup adapter, single	770079
Bluetooth printer 5.6	RSS flash Thermal printer 5.6 cm	770074
Bluetooth printer 8	RSS flash Thermal printer 8 cm	770077

Our **customer service team** is on hand to answer any questions you may have by telephone, via email or in a personal consultation. For details on **your local sales office and regional free phone numbers, visit www.nal-vonminden.com.** 





Headquarters **nal von minden GmbH** · Carl-Zeiss-Strasse 12 · 47445 Moers · Germany Chief Executive Officers: Sandra von Minden · Roland Meißner · Thomas Zander Commercial reg. Kleve · HRB 5679 Tel.: +49 941 290 10-0 · Fax: +49 941 290 10-50

info@nal-vonminden.com www.nal-vonminden.com